Avansert Nevromonitorering

- Mikrosensor teknologi
- Parenchymal
- ICP-Temp-Vevsoksygenering-Microdialyse
- Ventrikkeldren med mikrosensor
- Kompatibelt med all pasientovervåkning
- Big data

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➤ About Raumedic

- 800 employees, 120 EUR Million revenue
- Products are available since 1996
- Worldwide distribution

➤ Quality / Production

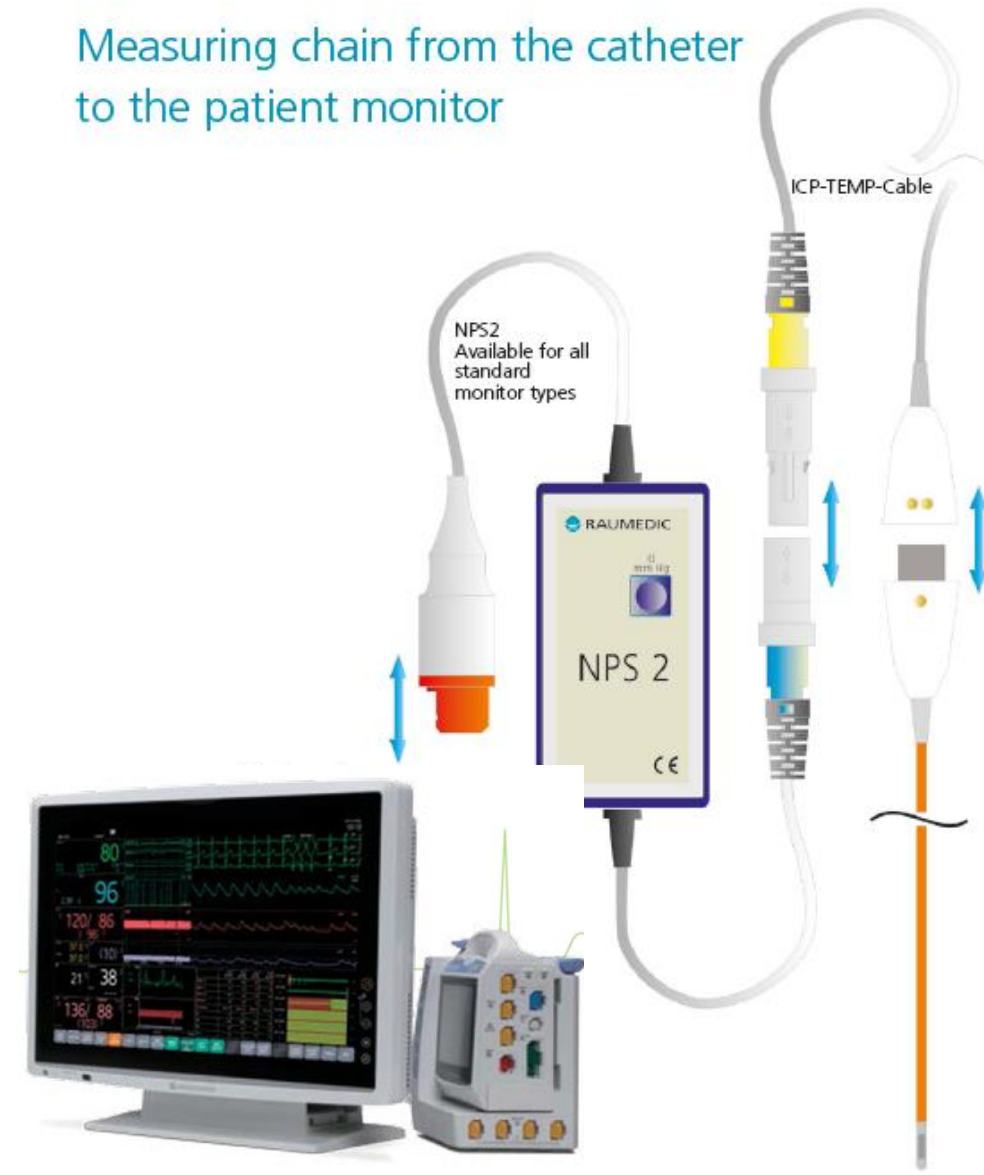
- Catheters are handmade
- 100% controll of every catheter
- Catheters are precalibrated in the plant
- Low drift
- Plug and Play (no intermediate monitor)
- Devices according to EMC 4 Standard

➤ Reference hospitals

- 75% market share in Germany
- University Hospital Regensburg, University Hospital John Cadcliff Oxford, Karolinska University Hospital

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
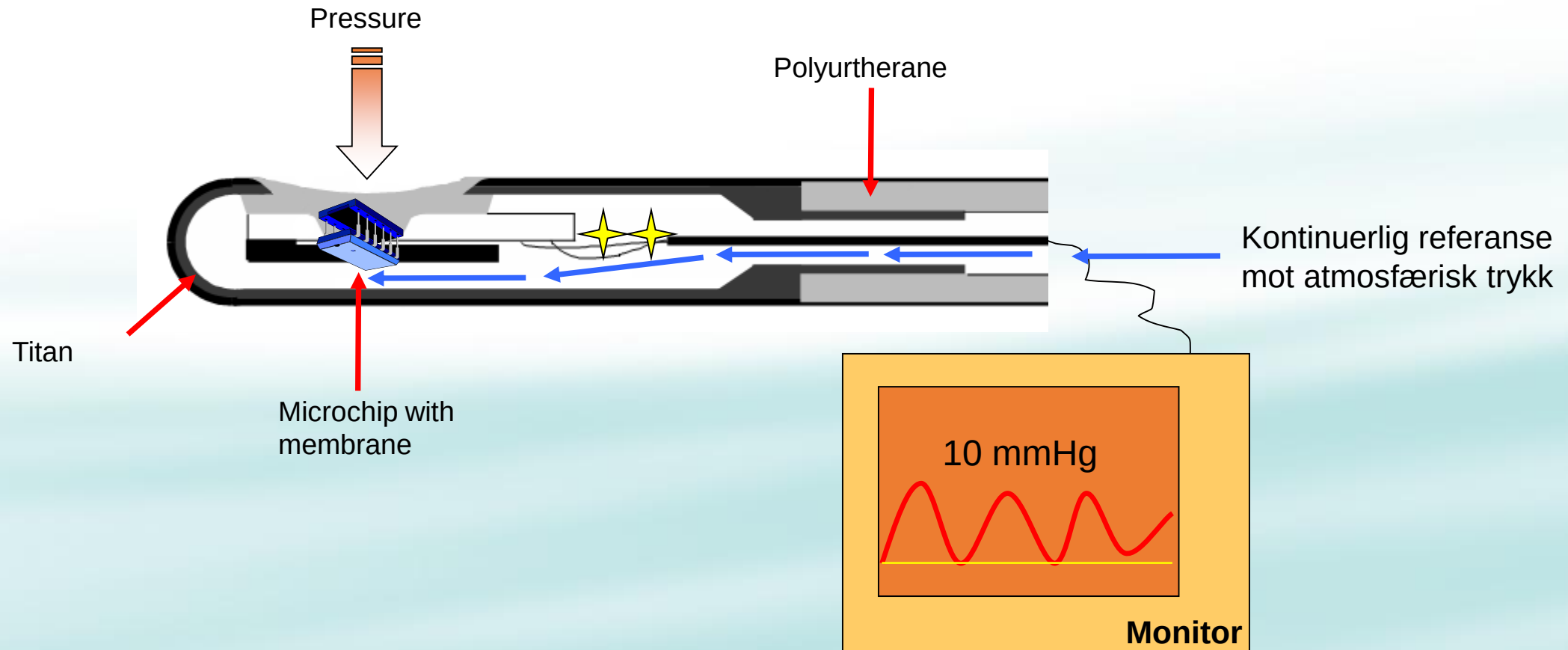
Measuring the ICP


- **RAUMEDIC uses microchip technology to produce high precision ICP measuring catheters:**

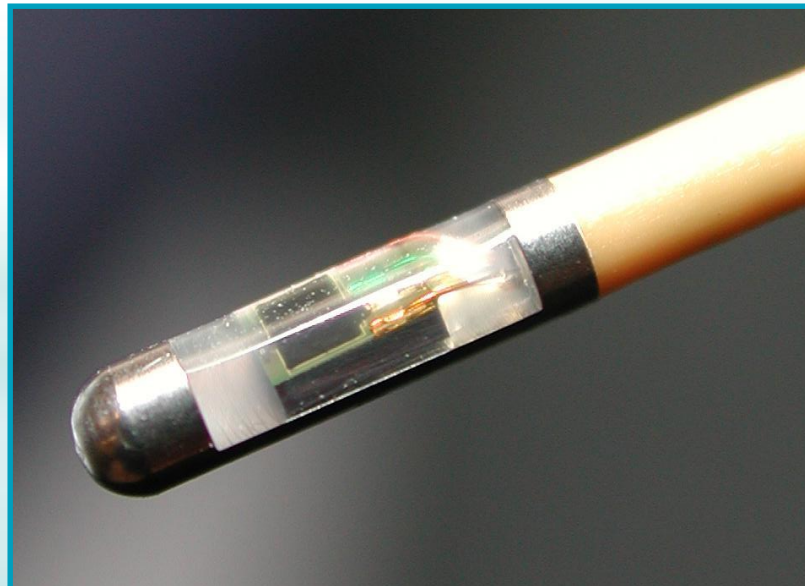How it works:
Differential pressure measurement and
Wheatstone Bridge circuit

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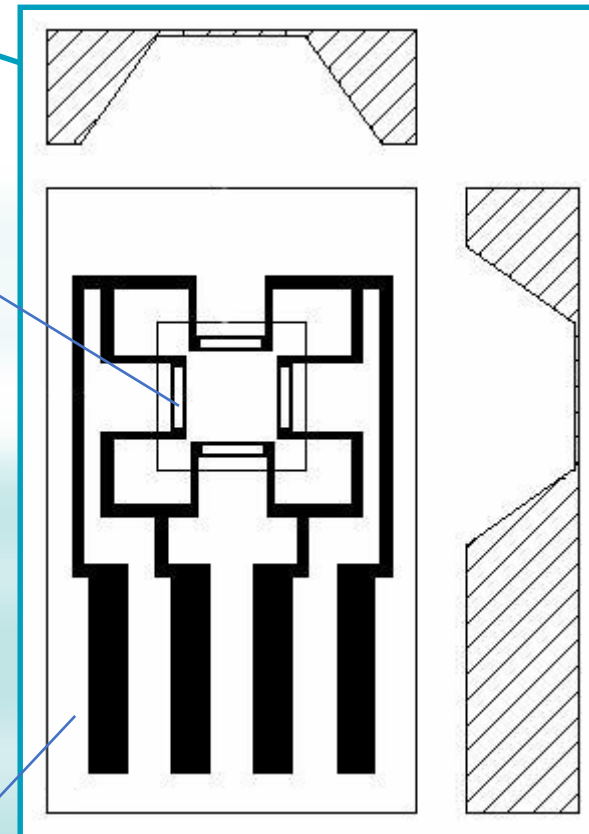Comparison of material properties of the disposable ventricular catheter

SEM* image of the RAUMEDIC PU ventricular catheters**:

Smooth surface structure of the RAUMEDIC catheter made of polyurethane

SEM* image of a standard ventricular catheter, typically of silicone

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RAUMEDIC – Spliceable Tunneling Sleeve

The Spliceable Tunneling Sleeve is suitable for routing the RAUMEDIC catheter and is intended for single use only.

Product	Article number
Spliceable Tunneling Sleeve CH8 (for parenchymal catheters)	090 506-002
Spliceable Tunneling Sleeve CH12 (for ventricular catheters and the NEUROVENT-PTO 2L)	090 717-001

Advantages:

- Sharpened trocar tip
- Chamfered sleeve for low resistance application
- Coloured tear-off strip for easy use
- Spliceable Tunneling Sleeve made of biocompatible, polymer material – in-vitro tested according to DIN EN ISO 10993/1

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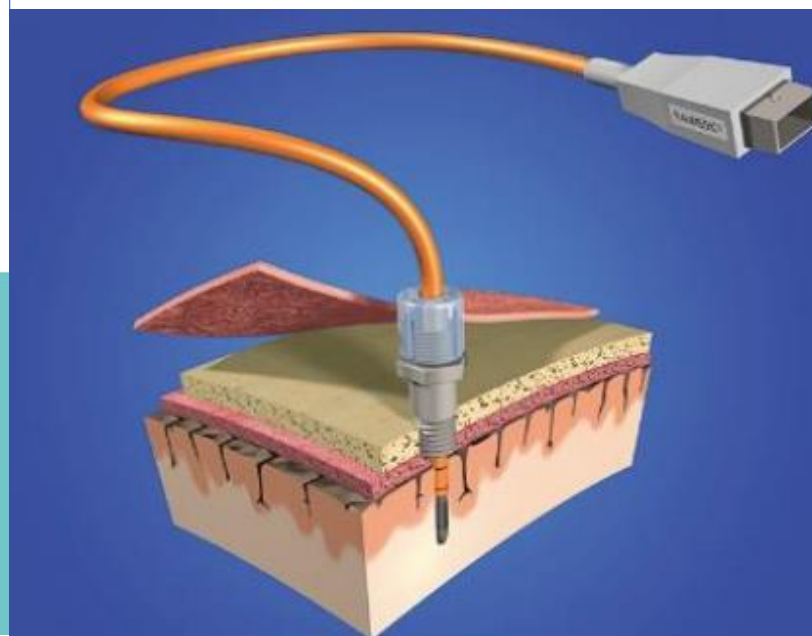
BOLT KIT PTO 2L

Two lumen BOLT for safe and functional implantation of the NEUROVENT-PTO 2L and a microdialysis catheter

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BOLT-DRILL KIT components

BOLT KIT components:

- 1 Polymer screw with seal
- 2 Fixing cap
- 3 Dura opener
- 4 Screw-in tool

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