

## Performing a Urinalysis Strip Test and a hCG Cassette Test on the Clinitek Status® Connect System/Clinitek Status®+ Analyzer

READ THIS GUIDE BEFORE PERFORMING A TEST.  
FOR DETAILED INFORMATION, REFER TO THE OPERATOR'S GUIDE.  
*Test for Quality Control as directed on page 6. See information about CLIA waiver on page 6 (for US Only).*

### Urinalysis Strip Testing: Performing a Full Test

Test fresh urine samples within 2 hours of collection. Refrigerated urine samples must be brought to room temperature (20° to 30°C) before testing. Do not test urine that looks bloody or is not a normal color.

Before beginning the strip test, make sure you have the following items:

- Siemens Healthcare Diagnostics reagent test strip
- Urine specimen
- Paper towel to blot strip



#### STEP 1


#### Getting Started

- Turn ON the Clinitek Status®+ analyzer.
- Turn the table insert so reagent strip holder is facing up.
- Be sure the test table, the table insert, and the calibration bar are clean.



#### STEP 2

- Select Strip Test from Select Ready screen.
- Remove a reagent strip from the bottle and quickly replace the cap.

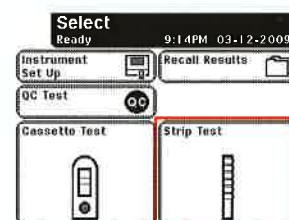

#### BIOHAZARD

Wear personal protective equipment. Use universal precautions. Refer to your Clinitek Status+ Operator's Manual for recommended precautions when working with biohazardous materials.

Abnormal results marked by "\*\*". Refer to package insert for product limitations, storage, and expiration date. Do not use reagent strip past the expiration date. If error occurs, follow screen prompts or consult the Clinitek Status+ Operator's Manual. For assistance, or if the system does not work as expected, call 877-229-3711.